


CLM-110-HU0504 Kit Components

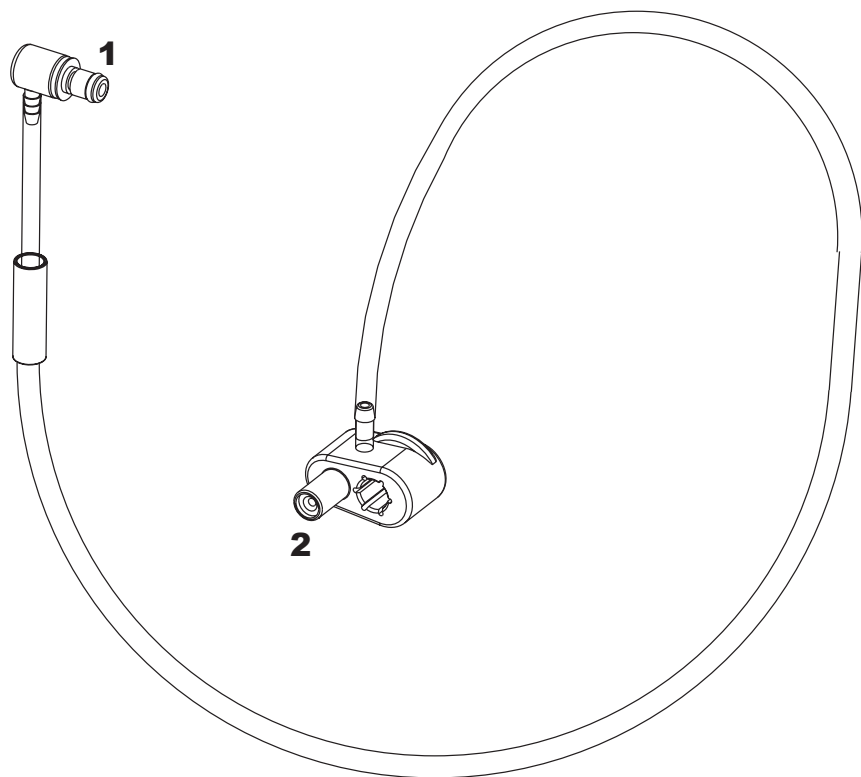

1. Basin CPC Connector
2. Air Probe Connector

WARNINGS AND CAUTIONS

Hookups must be periodically inspected to ensure that their functionality is not impaired. Check for damage/worn connectors, missing/damaged components, kinked/blocked tubing and other damage.

DO NOT AUTOCLAVE.


Check to make sure that connections remain firmly attached during the disinfection cycle and until the completion of the cycle to ensure proper high level disinfection.

Refer to endoscope manufacturer's instruction guide for more information.

NOTE: If you have any questions regarding this hookup please contact your service technician or contact Medivators Customer Support at 1.800.444.4729 or 763.553.3300.

www.minntech.com/medivators

CLM-110-HU0504 Kit Components

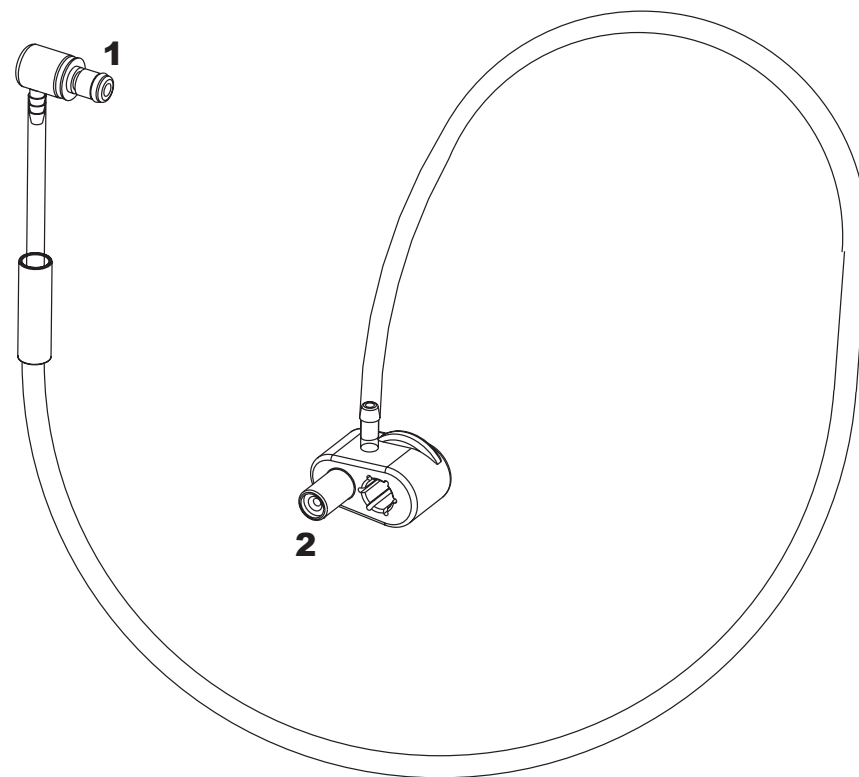

1. Basin CPC Connector
2. Air Probe Connector

WARNINGS AND CAUTIONS

Hookups must be periodically inspected to ensure that their functionality is not impaired. Check for damage/worn connectors, missing/damaged components, kinked/blocked tubing and other damage.

DO NOT AUTOCLAVE.

Check to make sure that connections remain firmly attached during the disinfection cycle and until the completion of the cycle to ensure proper high level disinfection.

Refer to endoscope manufacturer's instruction guide for more information.

NOTE: If you have any questions regarding this hookup please contact your service technician or contact Medivators Customer Support at 1.800.444.4729 or 763.553.3300.

www.minntech.com/medivators



Manufactured in the USA by:
MEDIVATORS
 REPROCESSING SYSTEMS
 A Minntech Corporation Business Group

Minntech Corporation
 14605 28th Avenue North
 Minneapolis, MN 55447 USA
 Phone: +763.553.3300
 Toll Free: 1.800.328.3340
 Fax: +763.553.3367
 website:www.minntech.com

Minntech BV
 Sourethweg 11
 6422PC Heerlen
 The Netherlands
 Phone: +31.45.5.471.471
 Fax: +31.45.5.429.695
 email:info@minntechbv.com

Minntech Asia/Pacific Pte Ltd
 No.1A International Business Park
 #05-01 Singapore 609933
 Tel: +65 6227 9698
 Fax: +65 6225 6848

Minntech Beijing Representative
 AVIC PLAZA, Room 1508B
 No. 10 Yi Dongsanhuan Zhonglu,
 Beijing 100022, China
 Tel: +8610.6567.8446
 Fax: +8610.6567.8445



Manufactured in the USA by:
MEDIVATORS
 REPROCESSING SYSTEMS
 A Minntech Corporation Business Group

Minntech Corporation
 14605 28th Avenue North
 Minneapolis, MN 55447 USA
 Phone: +763.553.3300
 Toll Free: 1.800.328.3340
 Fax: +763.553.3367
 website:www.minntech.com

Minntech BV
 Sourethweg 11
 6422PC Heerlen
 The Netherlands
 Phone: +31.45.5.471.471
 Fax: +31.45.5.429.695
 email:info@minntechbv.com

Minntech Asia/Pacific Pte Ltd
 No.1A International Business Park
 #05-01 Singapore 609933
 Tel: +65 6227 9698
 Fax: +65 6225 6848

Minntech Beijing Representative
 AVIC PLAZA, Room 1508B
 No. 10 Yi Dongsanhuan Zhonglu,
 Beijing 100022, China
 Tel: +8610.6567.8446
 Fax: +8610.6567.8445

