

# Product Bulletin

## For Distribution To:

- International Distributors
- International Management
- U.S. Field Sales & Clinical Specialists
- U.S. Technical & Field Service

Date: January 16, 2013

## RE: **Advantage® Plus Waterline Disinfection (WLD) cycle time and chemistry reduced**

The Advantage Plus waterline disinfection cycle duration and required doses of chemistry have been reduced to two hours and two doses. The new cycle name is S-WLDisinfect-2hr.

A new waterline disinfection parameter set was created that reduces the total cycle time for the WLD from 6 hours to 2 hours. The cycle now performs successful self-disinfection in 2 hours by removing the right basin disinfection step performed at the beginning of the cycle. This new parameter set works only with software version 3.1.1.0.

Software version 3.1.1.0 includes a reduction in the amount of Rapicide PA required for the cycle, from three doses to two doses. This new software will be implemented into production of new Advantage Plus units starting with serial number 68363908.

The new waterline disinfection cycle duration of 2 hours with 2 doses of chemistry has been validated to be as effective as the previous cycle titled "S-disinfect-4hr".

The new cycle, S-WLDisinfect-2hr is found in the WLD folder under Parameter Sets in the Management program. Also located in this folder is cycle S-WLDRinse, which is used to recover from failed WLD.

For additional information please contact Customer Support at #800-444-4729 or your local MEDIVATORS Representative.



MEDIVATORS Inc. Headquarters  
14605 28th Avenue North  
Minneapolis, MN  
55447-4822 USA  
Tel: +1.800.444.4729  
Fax: +763.553.3387

MEDIVATORS BV  
Sourthweg 11  
6422PC Heerlen  
The Netherlands  
Tel: +31.45.5.471.471  
Fax: +31.45.5.429.695

MEDIVATORS Asia/Pacific Pte Ltd  
1 A International Business Park  
#05-01 Singapore 609933  
Tel: +65.6227.9698  
Fax: +65.6225.6848

MEDIVATORS Beijing Representative Office  
Room 708, 7th Floor Kaiheng Center, Block B  
No. 2 Chaoyangmennei Street  
Dongcheng District, Beijing China 100010  
Tel: +8610.6567.8446  
Fax: +8610.6567.8445

[www.medivators.com](http://www.medivators.com)