# Rapicide<sup>®</sup> PA

**High Level Disinfectant  
for Endoscope Reprocessing**

## DIRECTIONS FOR USE

---

**KEEP OUT OF REACH OF CHILDREN**  
**DANGER**

**WARNING**

*For endoscope reprocessing in an automated endoscope reprocessor only. Part A and B can not be used separately.*

**PACKAGING:**

4 Bottles per Case (20 Liters)

2 Bottles of Part A X 5 Liters

2 Bottles of Part B X 5 Liters

**Catalog #: ML02-0116**

---

Distributed in Canada by:  
**CARSEN MEDICAL INC.**

14-151 Amber Street  
Markham, Ontario, Canada L3R3B3  
Phone: 866.677.4121

## DESCRIPTION OF ACTIVATED SOLUTION

### Part A & Part B –

Concentrated high level disinfecting solution, with sporicidal activity when used in an automated endoscope reprocessing system.

### PRODUCT EFFICACY:

For high level disinfection of endoscopes. Contact time five (5) minutes at 30°C. Sporicidal efficacy in 5 hours at 30°C.

### DIRECTIONS FOR USE:

Connect one bottle of Part A and one bottle of Part B to an automated endoscope reprocessor per user manual instructions. Ensure the automated endoscope reprocessor mixes the correct volumes of water, Part A and Part B to achieve at least 850 ppm of peracetic acid (Minimum Recommended Concentration). Check the use solution with a Rapicide PA Test Strip to ensure that it is above MRC.

### COMPATIBILITY:

The activated solution has been shown to be highly compatible with the materials of endoscopes and the Automated Endoscope Reprocessor. Data is on file with the manufacturer.

### STORAGE:

Keep away from children. Keep in a dry place and store at room temperature away from heating sources. Expiration date applies to properly preserved unused product. Do not use after expiration date. The product may be used for three (3) weeks after the containers are opened. DO NOT use after that time has expired.

The activated and diluted (A+B) product does not require additional specific safety measures for people or the environment. After use the spent solutions must be disposed of following Local, Provincial and Federal Regulations.

---

## MEDIVATORS<sup>®</sup>

Manufactured in the USA by:  
MINNTECH CORPORATION  
14605 28th Avenue North  
Minneapolis, MN 55447 USA  
Toll Free: (800) 444-4729  
FAX: (763) 553-2499

# Rapicide<sup>®</sup> PA

High Level Disinfectant  
for Endoscope Reprocessing

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**MEDIVATORS<sup>®</sup>**

Fabriqué aux Etats-Unis par:

MINNTECH CORPORATION  
14605 28th Avenue North  
Minneapolis, MN 55447 USA  
Toll Free: (800) 444-4729  
FAX: (763) 553-2499

© 2011, MINNTECH CORPORATION  
50097-381/B