

## QUICK REFERENCE GUIDE

# RAPICIDE™ PA

## Test Strips

RAPICIDE™ PA Test Strips are used for testing the concentration of RAPICIDE™ PA High Level Disinfectant (HLD)\*. RAPICIDE PA HLD is available for use with the ADVANTAGE PLUS™ Pass-Thru Automated Endoscope Reprocessor (AER), ADVANTAGE PLUS™ AER and DSD EDGE™ AER.



## TEST STRIP FEATURES

- Fast 30-second reaction time
- Definitive pass/fail color identification comparison chart
- 850 parts per million (ppm) minimum recommended concentration (MRC) level
- Shelf life is one year unopened and four months once opened
- Large, easy-to-read indicator pad

## PACKAGING

- 100 test strips per bottle, 2 bottles per package
- Part number: #ML02-0118

## CUSTOMER SUPPORT

- For questions or to place an order, please contact Customer Support at 1-800-444-4729.

## DIRECTIONS FOR USE

Read the entire RAPICIDE PA Test Strips package instructions for important product information and proper usage instructions. Refer to the ADVANTAGE PLUS Pass-Thru AER, ADVANTAGE PLUS AER or DSD EDGE AER user manuals for complete instructions on how to take a sample of the disinfectant use solution. It is recommended that the RAPICIDE PA HLD solution be tested with each endoscope reprocessing cycle to verify disinfectant is above 850ppm MRC.

- Open the RAPICIDE PA Test Strips bottle and remove one test strip. Close the bottle.
- For the ADVANTAGE PLUS Pass-Thru AER - Dip the test strip indicator pad into the sample port for one second and immediately remove. Gently shake off excess solution.
- For the ADVANTAGE PLUS AER – Use the sampling cups to collect the solution sample from the reprocessor. Dip the test strip indicator pad into the solution in the sampling cup for one second and immediately remove. Gently shake off excess solution.
- For the DSD EDGE AER - Dip the test strip indicator pad into the sample port (located inside the basin) for one second and immediately remove. Gently shake off excess solution.
- Wait 30 seconds and then compare the test strip color with the color chart located on the side of the test strip bottle to determine peracetic acid ppm is above MRC.

### Test Strip Comparison Chart



[www.medivators.com](http://www.medivators.com)

\*RAPICIDE PA Test Strips are not compatible with RAPICIDE PA Ready-to-Use HLD. ADVANTAGE PLUS™, DSD EDGE™ and RAPICIDE™ are trademarks of Medivators Inc.

© 2018 Medivators Inc. P/N 50097-532/D

 **MEDIVATORS**  
A Cantel Medical Company