

MEDIVATORS®
RAPICIDE®
High Level Disinfectant Family



MEDIVATORS® RAPICIDE®

High Level Disinfectant Family

RAPICIDE PA High-Level Disinfectant


RAPICIDE PA High-Level Disinfectant is a single-use, peracetic acid-based solution that is environmentally responsible and requires no neutralization. It is designed for use in the MEDIVATORS® ADVANTAGE PLUS® or MEDIVATORS DSD EDGE® Automated Endoscope Reprocessors.

- 5-minute contact time
- Used at 20°C
- Proven endoscope compatibility – no increased wear on endoscope materials at this low temperature
- Single-use chemistry; aldehyde-free
- Requires only 1 rinse, reducing the total cycle time
- Environmentally responsible – breaks down into water and vinegar
- Minimum recommended concentration (MRC) is 850ppm of PAA

RAPICIDE PA Test Strips

RAPICIDE PA Test Strips are used to measure the minimum recommended concentration (MRC) of RAPICIDE PA High-Level Disinfectant.

- Color chart to confirm concentration is shown on test strip bottle
- 1 second dip time; 30 second read time

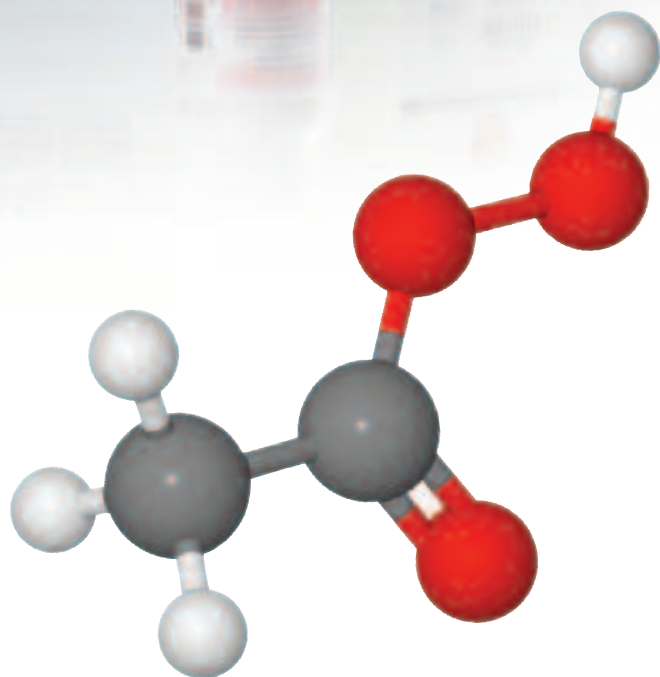

For almost 10 years and 2,700+ users, the RAPICIDE® Disinfectant Family is THE choice for high-level disinfection.

RAPICIDE Glut High-Level Disinfectant and Sterilant

RAPICIDE Glut High-Level Disinfectant is a fast-acting, non-staining solution that requires no mixing or activation. It is a reusable glutaraldehyde-based disinfectant designed for use on flexible endoscopes in automated endoscope reprocessors.

- 5-minute contact time
- Used at 35°C
- Effectively kills TB, hepatitis viruses and C. diff*
- Up to 28-day reuse life
- Improved efficiency; requires only 2 rinses
- Ready-to-use, no mixing or activation required
- Active chemical is 2.5% glutaraldehyde
- Minimum recommended concentration (MRC) is 1.5% glutaraldehyde

RAPICIDE Glut Test Strips

RAPICIDE Glut Test Strips are used to measure the minimum recommended concentration (MRC) of RAPICIDE High-Level Disinfectant and Sterilant.

- Color chart to confirm concentration is shown on test strip bottle
- 3 second dip time; 75 second read time



RAPICIDE OPA/28 High-Level Disinfectant

RAPICIDE OPA/28 High-Level Disinfectant works faster and lasts longer for superior reprocessing of heat-sensitive, semi-critical medical devices. RAPICIDE OPA/28 High-Level Disinfectant features the fastest disinfection time, twice the reuse period of other OPA brands and guaranteed materials compatibility for the ultimate combination of safety, convenience and value.

- 5-minute contact time at 20°C
- Up to 28-day reuse life
- Effectively inactivates TB, MRSA and VRE*, HIV² and Hepatitis Viruses
- Requires only 2 rinses in a MEDIVATORS® reprocessor; 3 rinses if used manually
- Active chemical is 0.575% ortho-phthalaldehyde (OPA)
- Minimum recommended concentration (MRC) is 0.35% OPA
- Available for manual application

RAPICIDE OPA/28 Test Strips

RAPICIDE OPA/28 Test Strips are used to measure the minimum recommended concentration (MRC) of RAPICIDE OPA/28 High-Level Disinfectant.

- Color chart to confirm concentration is shown on test strip bottle
- 3 second dip time; 90 second read time



*Technical data available upon request

RAPICIDE™ Spill Kits

The RAPICIDE Spill Kits are available for use with the MEDIVATORS family of RAPICIDE® products. The spill kits allow customers a quick and easy way to neutralize spills at their facility. The kit includes the appropriate neutralizer and the Personal Protective Equipment (PPE) necessary to safely clean up a spill or leak.



The RAPICIDE Spill Kit can be used for RAPICIDE® Glut High-Level Disinfectant and RAPICIDE OPA/28 High-Level Disinfectant. It includes:

- NEUTRA-HYDE® glycine neutralizer
- NEUTRA-HYDE SDS
- RAPICIDE glut SDS
- RAPICIDE OPA-28 SDS
- Bouffant cap
- Nitrile gloves
- Impervious gown
- Shoe covers
- Safety goggles with face shield
- Large absorbent mats



The RAPICIDE PA Spill Kit can be used to clean up RAPICIDE PA High-Level Disinfectant.

It includes:

- NEUTRA-Ox sodium bicarbonate neutralizer
- NEUTRA-Ox MSDS
- RAPICIDE PA MSDS
- Bouffant cap
- Nitrile gloves
- Impervious gown
- Shoe covers
- Safety goggles with face shield
- Mixing knife

ORDERING INFORMATION

RAPICIDE Disinfectant and Sterilant

Part number	Description
ML02-0059	RAPICIDE Glut High-Level Disinfectant and Sterilant (4 one-gallon bottles/case)
ML02-0120	RAPICIDE Glut Test Strips (2 bottles/pack, 60 strips/ bottle)

RAPICIDE OPA/28 High-Level Disinfectant

Part number	Description
ML02-0132	RAPICIDE OPA/28 High-Level Disinfectant (4 one-gallon bottles/case)
ML02-0136	RAPICIDE OPA/28 Test Strips (2 bottles/pack, 50 strips/ bottle)

RAPICIDE PA High-Level Disinfectant

Part number	Description
ML02-0115	RAPICIDE PA High-Level Disinfectant (2 each 5 liter bottles Part A and Part B/case)
ML02-0129	RAPICIDE PA Test Strips (2 bottles/pack, 100 strips/ bottle)

High-Level Disinfectant Spill Kits

Part number	Description
78400-973	For RAPICIDE Glut and RAPICIDE OPA/28 (1 Kit)
78400-974	For RAPICIDE PA (1 Kit)
DNS-1002	NEUTRA-HYDE Neutralizer
DNS-1006	NEUTRA-Ox Neutralizer

MEDIVATORS®, RAPICIDE®, ADVANTAGE PLUS®, DSD EDGE® and NEUTRA-HYDE® are registered trademarks of Medivators Inc.



Medivators Inc.
14605 28th Avenue North
Minneapolis, MN
55447-4822 USA
Tel: +1.763.553.3300
Fax: +1.763.553.3387

Medivators BV
Sourethweg 11
6422 PC Heerlen
The Netherlands
Tel: +31.45.5.471.471
Fax: +31.45.5.429.595

www.medivators.com

Cantel Medical (UK) Limited
Wolseley House
Staffordshire Technology Park
Beaconside, Stafford ST18 0GA
United Kingdom
Tel: +44 (0) 1785.782420
Fax: +44 (0) 1785.782421

Cantel Medical Asia/Pacific Pte. Ltd.
1A International Business Park
#05-01 Singapore 609933
Tel: +65.6227.9698
Fax: +65.6225.6848

Medivators Inc. Beijing Representative Office
Room 1801, Floor 18th, Tower A,
Beijing Marriott Hotel, Office Building
No. 7, Jianguomen South Avenue,
Dongcheng District, Beijing 100005 China
Tel: +86 10 65204039