

FEATURES

- Low foaming, neutral pH detergent specifically formulated for use with medical instruments including immersible endoscopes, related accessories, and surgical instruments.
- Patented formula will rapidly penetrate and solubilize blood, proteins, cellular mucosal debris, carbohydrates, lipids and mucopolysaccharides.
- Laboratory proven to effectively remove biological and organic residues from a variety of medical instruments and surfaces.
- INTERCEPT Detergent may reduce the film build-up that can occur from the use of non-glutaraldehyde high-level disinfectants on the devices or accessories.

**WARNING**

INTERCEPT DETERGENT
IS NOT A DISINFECTANT
AND IS NOT INTENDED
TO DISINFECT
MEDICAL INSTRUMENTS.



READ PACKAGE INSERT AND
SDS FOR ADDITIONAL
PRECAUTIONARY
STATEMENTS. SEE PACKAGE
INSERT FOR ADDITIONAL
LANGUAGE TRANSLATIONS.

LOT:**EXP:**

Intercept™

Detergent

A low foaming, neutral pH detergent for
cleaning of immersible medical instruments.

NET CONTENT
1 U.S. Gallon
(3.8 Liters)



MEDIVATORS
A Cantel Medical Company

Manufactured in the USA by:



Medivators Inc.
14605 28th Avenue North
Minneapolis, MN 55447 USA
Toll Free: +1.800.444.4729

www.medivators.com

REF ML02-0106


KEEP OUT OF REACH OF CHILDREN CAUTION

CONTENTS: sodium-hydroxide

HAZARD & PRECAUTIONARY STATEMENTS: Causes skin irritation. Causes serious eye irritation. Wash hands thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. If on skin: Wash with plenty of water. Take off contaminated clothing and wash it before reuse. If in eyes: Rinse cautiously with water for several minutes. If eye or skin irritation persists: Get medical advice/attention.

In case of ingestion, DO NOT induce vomiting. Contact CHEMTREC for emergency medical treatment at 1-800-424-9300.

STORAGE & DISPOSAL: Store at room temperature 15°C-25°C (59°F-77°F). INTERCEPT Detergent has a two (2) year shelf life from date of manufacture, open or unopened as long as it is stored according to labeled instructions. Discard unused solution according to local, state and federal regulations. DO NOT reuse container.

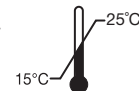

DIRECTIONS FOR USE

For cleaning of fully immersible endoscopes, related accessories, surgical instruments, and other apparatus where blood, mucus, protein or other hard to remove soils are encountered, use INTERCEPT Detergent at 1/3 oz./gal of water (0.25% use concentration) with one full stroke of the hand-pump (1 oz.) to three gallons of water. For best manual cleaning results, mix INTERCEPT Detergent with cool to warm water (20°C - 35°C) (68°F - 95°F) and ensure a minimum contact time of (1) one minute. Rinse all surfaces and internal channels thoroughly with water. For use in automated washers, follow washer manufacturer's recommendations. Do not mix INTERCEPT Detergent with bleach.

INTERCEPT™ is a trademark of Medivators Inc.
© 2018 Medivators Inc. 67199-625/E